

## **Digital Display**

Displaying the Impulse current from 0.20 to 5.0mA Pulse LED (Yellow) indicates that the device is ready.

## **Battery LED**

Indicates low battery & battery should be replaced.

## Frequency

The frequency can be toggled between 1Hz and 2Hz by holding the concerned key for 2 sec. The default frequency is 1Hz.

**Beep Tone** is heard briefly during Self Test and after that it is triggered only when the circuit is closed.





## **Specifications:**

Compact

User-friendly & light weight
Constant current stimulator – monophasic
Audio visual indication for pulse delivery
Large 7 segment LED display
Battery low indicator

Pulse Amplitude : 0.2mA to 5.0mA

Pulse Width : 0.1m sec

Pulse Frequency: 1Hz or 2 Hz switchable (Default 1 Hz)

Impedance : 0 to 5K Ohms

Display : 3 Digit 7 segment large display

Resolution : 0.01mA or 0.1mA

Battery : 9V, Type 6LR-61 or 6F22

Indication : Red LED flashing indicates low battery

Yellow LED flashing with beep sound for delivery of each pulse when the

circuit is closed





Manufactured By:

Medilogix Medical Technologies Pvt Ltd # 127 (Old # 58), Vedhachalam Street, Vasudevan Nagar, Ashok Nagar, Chennai - 600 083. Ph: 044 2471 7220, 93452 65450, Email: contact@medilogix.in www.medilogix.in